

Delivering MetaTherapy® Treatment through The MetaNeb® System



Enhancing outcomes for
patients and their caregivers:


Hill-Rom

MetaTherapy® Treatment

MetaTherapy® Treatment maximizes efficiency for patients and clinicians by combining lung expansion, secretion clearance and aerosol delivery into a single integrated therapy cycle. May reduce therapy time to just 10 minutes,* and eliminate the need to switch devices.

Lung Expansion, Secretion Clearance and Aerosol Delivery in a Single Therapy Cycle

Only from Hill-Rom

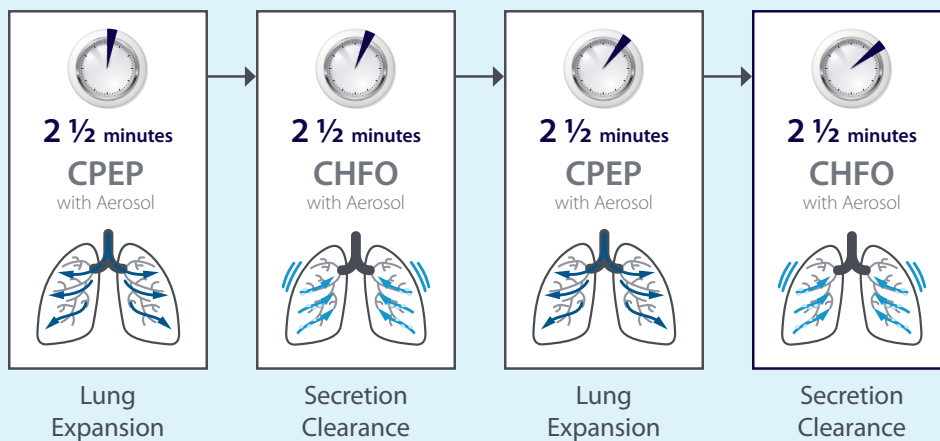

**10 minutes
Total***

Alternating cycles of CPEP and CHFO combined with aerosol delivery helps maximize therapy effectiveness.

Benefits of MetaTherapy® Treatment

- May save time for patient and clinician
- Introduces CHFO therapy immediately after lungs have been expanded through CPEP
- Integrates aerosol delivery of medications for maximum efficiency

* Therapy times may differ depending on patient needs and ordered therapies.



Trusted Solutions from Hill-Rom

Only Hill-Rom offers the integrated convenience and efficacy of MetaTherapy® Treatment delivered through The MetaNeb® System. Hill-Rom Respiratory Care is a division of Hill-Rom, a leading worldwide manufacturer and provider of medical technologies and related services for the health care industry, with over 80 years of innovation and excellence. Hill-Rom — Enhancing Outcomes for Patients and Their Caregivers.®

The MetaNeb® System

The Convenience of MetaTherapy® Treatment Delivered through a Single Unit

- Simple “connect and go” design
- Single patient use (SPU) circuit includes mouthpiece, nebulizer, mask/trach adapter, and in-line ventilator setup
- Quickly switch between therapy cycles
- Easily adjustable flow, pressure and percussive rate

Delivery Options

- In-line ventilation
- Mouth piece
- Face mask
- Tracheostomy

Conditions for MetaTherapy® Treatment

- Chronic obstructive pulmonary disease (COPD)
- Post-operative airway management
- Bronchiectasis
- Neuromuscular disorders
- Cystic fibrosis
- Asthma
- Emphysema
- Reversal of atelectasis
- Chest wall trauma

Hospital-to-Home Continuity of Care

Hospitals are looking for effective continuum of care solutions to get patients healthy in the acute care setting and stay healthy after they return home. The MetaNeb® System can be part of that solution, along with another proven Hill-Rom solution: The Vest® Airway Clearance System.



Hospital Care

Patients receive therapy with The MetaNeb® System or The Vest® System.

1

Respiratory Care

2

Stabilization

3

Discharge



Home Care

Improve airway clearance to reduce the risk of re-hospitalization with home use of The Vest® System.¹

The MetaNeb® System and The Vest® Airway Clearance System both are proven therapies for airway clearance.

The Vest® Airway Clearance System

The Vest® System is designed to assist patients in mobilizing retained secretions that can otherwise contribute to increased rates of respiratory infection, reduced lung function and costly re-hospitalization.

Designed for both hospital and in-home use, The Vest® System can be an effective home-care regimen for patients who have been using The MetaNeb® System in the hospital.

To learn more:

www.respiratorycare.hill-rom.com



The MetaNeb® System



Part Number	Description	Product
PMN4	The MetaNeb® 4.0 System	Includes: System Controller, Mobile Stand, Source Gas Hose, 2 SPU circuits, and User Manual
Single Patient Use Circuits		
C20000N	SPU circuit with nebulizer included	1 circuit
PC20005N	SPU circuit with nebulizer included	5 circuits / kit
Replacement Parts		
M06330S	Bracket Arm	Bracket Arm Assembly
182024S	Caster Kit	Caster Kit (2 locking casters and 3 non locking casters)
178176	Circuit Kit Adapter	Adapter Kit (includes occlusion ring, 22x22 , 22x15, mouthpiece)
165860	Filter Screen Kit	O-ring (1 each), Filter Screen (1 each) and Spring (1 each)
181502S	Manometer Face Plate	Bezel (1 each)
C20019S	O-Ring Kit	O-Rings (4 each)
173538	Oxygen Supply Hose	Supply Hose (1 each)
181500S	Control Knob Kit	Replacement Knobs for the Controller
171700	Selector Ring	Selector Ring (1 each)
174569	Silicone Adapter	Silicone Adapter (1 each)
177895	Stand	Complete Stand and Pole

NOTE: If you have a different model than shown and need ordering information, please contact your sales representative or customer service at 800-426-4224.

1. This report was prepared by Milliman on January 16, 2012.
Full report available upon request.

The Vest® System is offered in the home by Advanced Respiratory, Inc., a Hill-Rom company. MetaNeb® is a registered trademark of Comedica, Inc. Enhancing Outcomes for Patients and Their Caregivers is a registered trademark of Hill-Rom Services, Inc.

Hill-Rom reserves the right to make changes without notice in design, specifications and models. The only warranty Hill-Rom makes is the express written warranty extended on the sale or rental of its products.

©2016 Hill-Rom Services PTE Ltd. ALL RIGHTS RESERVED.
165667 rev 5 21-DEC-2016 ENG – US

For further information about this product or a service, please contact your local Hill-Rom representative or visit our webpage:

800-426-4224

www.respiratorycare.hill-rom.com

Enhancing outcomes for
patients and their caregivers:

Hill-Rom